


# BinaxNOW Overview



BinaxNOW is an Antigen test for COVID-19 produced by Abbott Labs.

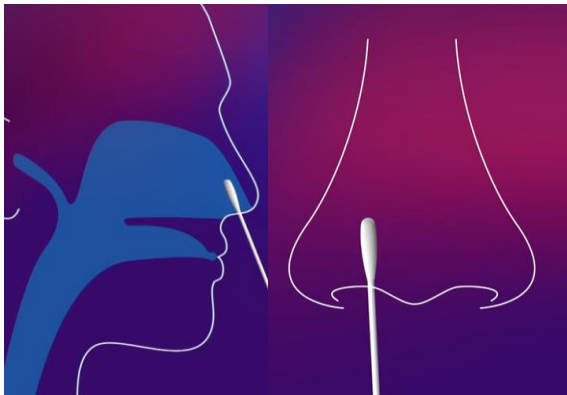

Remove sterile packaging

Gently insert swab into the patient's nostril until resistance is met, usually less than 1 inch

Rotate swab 5 times or more against nasal wall

Repeat sample collection with the same swab in the other nostril



Remove test card from packaging and lay flat

Open the test card and drop 6 drops of reagent into the TOP hole of the test card



Insert swab into the bottom hole and firmly push upward until the tip is visible in the top hole

Rotate swab 3 times clockwise and do NOT REMOVE THE SWAB




Peel off adhesive tape and close the card pushing firmly

The swab should remain inside of the test card while it processes the sample

# BinaxNOW Overview



Wait 15 minutes but no more than 30 minutes to read results

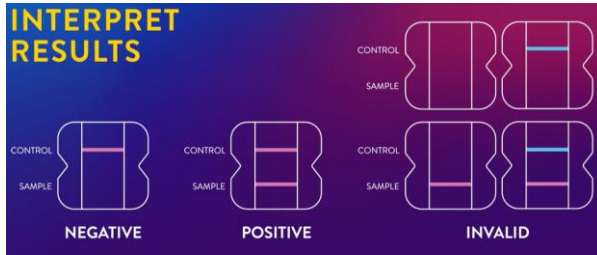

Interpret results and input into spreadsheet

\*Photos provided by  
<https://www.globalpointofcare.abbott/en/product-details/navica-binaxnow-covid-19-us.html>